

Technical specs

M1200VBC_ - M1200VB

Name:

BOX OF 10 DISPOSABLE HALF-MASKS FFP2 WITH VALVE - VERTICAL FOLDING

Colour:

White



CE 0194

Description :

Non woven synthetic fibre facial filtering part FFP2. Nose clip for adjustment. Foam reinforcement under nose clip. High performance exhalation valve. Vertical folding. Box of 10.

Materials:

Mask: non-woven synthetic.

Straps : Rubber

Instructions for use:

This mask is classified in accordance with its filtering capacity and maximum total inward leakage.

FFP2 : Protective mask against solid and liquid particles of average toxicity.

Can be used for concentrations up to 10 X of the Average Exposure Value (AEV)

*NR : Filter half-mask against particles not to be used for more than single shift.

*D : Mask resistant to filling in time (test by Dolomite dust) improving comfort of use.

Filtering capacity of the filter: 94% - Nominal Protection Factor (NPF) : 12.5

Only take the mask out of its packaging just before you are going to use it. Only use when the concentration of contaminating agents does not pose an immediate risk to life or to health, and only in observance of the applicable health and safety standards.

Before you use the mask, check that it is undamaged, that its expiry date has not been reached and that the protection class (FFP2) is appropriate for the product used and its concentration. Throw away after each use.

Limits to use:

The user must have adequate training in the wearing of the mask and must have read the instructions before using it. Use the mask in well-ventilated areas, where there is no lack of oxygen (> 20%) and where there is no explosive atmosphere. The mask must be positioned correctly in order to provide the expected level of protection. Do not use if facial hair (beard, sideburns) prevent the mask from being fully sealed, and leakage is detected. Leave the area immediately if you experience breathing problems and/or dizziness or faintness. Do not make any modifications to the mask, and discard it if there is any damage to it. Failure to observe these instructions for use may have adverse effects on the health of the user and expose him to serious or even life-threatening risks. This dust mask shall not be used for more than one shift (NR). Do not use other than for the scope of use defined in the above instructions for use. Contact with the skin may cause allergic reactions in people with sensitive skin. In this event, leave the area of risk, remove the mask and seek medical advice.

Instructions for storage:

Store in a cool dark place, in its original packaging (+2°C/+55°C - HR<75%).

Instructions for cleaning / maintenance:

This mask must only be used once (no maintenance).

Performances :

This mask complies with the European directive 89/686, notably regarding ergonomics, innocuousness, comfort, ventilation and flexibility, and with EN149:2001+A1:2009 type FFP2 *NR *D standard (nominal protective factor : 10 X AVE (Average

Technical specs

M1200VBC_ - M1200VB

Value of Exposure) for very toxic dust: lead, quartz, chromates).